Scarban®
Professional Scar Treatment

INSTRUCTIONS *for use*

SCAR TREATMENT AFTER C-SECTION

Because every scar needs care!

www.scarban.eu

SCAR TREATMENT AFTER C-SECTION

THE BEST TREATMENT FOR YOUR SCAR STARTS NOW!

Surgery like a caesarean section always comes with a scar. A scar that you obviously want to heal properly and reduce as much as possible.

With Scarban® C-Section silicone therapy you will provide your scar with the best possible treatment. Scarban® silicone sheets are the best choice, which is why they are recommended throughout the world.

Scarban® silicone sheets are highly effective and have been clinically proven to reduce scarring and complaints by creating a semi-occlusive layer on the skin.

This layer reduces water loss through the scar, which is one of the most important precursors for abnormal and disfiguring scar formation.

The result:

- Scar formation is reduced / minimised
- Significant relieve of complaints such as itching and pain
- Improved appearance of the scar, less redness

In conclusion, Scarban® C-section provides your skin with optimal support in the prevention of abnormal scarring.

So start your therapy today for a beneficial recovery!

TIPS

TIPS FOR USE

Please note that Scarban® silicone sheets are only intended for use after complete wound closure.

Start Scarban® silicone therapy after stitches are removed and scabs have disappeared.

For best results it's important to wear the Scarban® silicone sheet on the scar for at least 12 consecutive hours, every day.

Apply the Scarban® silicone sheet to clean, non-greasy skin. Also, do not use greasy creams or oil, as this will reduce the sheet's adhesive strength.

Scar recovery takes time. Please note that it can take up to 6 months, or even longer, for a scar to mature and recover optimally.

Clean the Scarban® silicone sheet daily. For example during your regular shower. This increases the sheet adhesive strength and, thus, its useful life.

Always keep your damaged skin out of the sun. Exposure to the sun can cause sunburn and permanent discolouration.

Scarban® silicone sheets are safe and effective at both new as old scars!

INDEX

Page

Introduction	2
Tips	3
Practical uses	4
EN	5
NL	6
FR	7
DE	8
ES	9
PT	10
IT	11
FI	12
PL	13
TR	14
AR	15
CN	16
TW	17
KO	18
JP	19

For further advise and practical usage see the instructions for use in your language in this booklet.

PRACTICAL USES

1

If necessary, the Scarban® silicone sheet can be cut to size. Ensure that the area 2 cm around the scar is covered with the silicone sheet.

2

For good adhesion and durability of the Scarban® silicone sheet it is advisable to remove any hair from the scar site. Clean and dry the skin thoroughly before applying the silicone sheet to the scar and the immediate, surrounding area.

3

Remove the protective film from the back of the Scarban® silicone sheet.

4

Stick the Scarban® silicone sheet to the scar.

5

If necessary, the Scarban® silicone sheet may be fixed in place with an adhesive or elastic bandage. The skin may be treated with a hydrating cream after removal of the silicone sheet.

6

Scarban® silicone sheets should be cleaned daily to maintain adhesion. Rub the silicone sheet with a few drops of Scarban® Silicone Cleaning Soap or a neutral soap.

7

Remove any dirt as much as possible using soap and lukewarm water. Rinse the silicone sheet well under lukewarm, running water.

8

After contact with soap and water the adhesive effect disappears and the Scarban® silicone sheet rolls up. Allow the silicone sheet to dry on a clean, dry surface with the adhesive side facing up. During drying, the silicone sheet automatically unrolls and the adhesive effect gradually returns.

Name:

Scarban® silicone sheets

Product description:

- Scarban® silicone sheets are a proven technology for treating scars (hypertrophic and keloid) and preventing abnormal scar development after surgery or an accident. The medical silicone sheet is self-adhesive (with the exception of the silicone sheets glove and finger), washable and can be used for a long period of time.
- Scarban® Silicone Cleaning Soap is a cleaning agent for silicone sheets.

Composition:

- Textile layer: polyamide, elastane
- Medical grade silicones

Properties:

- Scarban® silicone sheets ensure that scars become flatter, smoother and thinner and reduce complaints (like persistent redness, pulling of the scar, itching, restriction of joints movement, discomfort and cosmetic concerns).
- The silicone sheet provides a semi-occlusive (covering) layer on top of the skin, which results in increased hydration of the skin (epidermis). This increased hydration ensures that the damaged scar skin recovers more quickly and that functional objections (complaints) are reduced.
- Scarban® silicone sheets provide UV protection (UPF50) as standard.

Indications/applications:

Scarban® silicone sheets are suitable for:

- both old and new scars
- all skin types
- all ages
- treatment and prevention of abnormal scars (hypertrophic and keloid)
- treatment of burn scars
- treatment of surgical scars
- use under clothing and in compression therapy
- treatment of persistent functional complaints caused by the scar (persistent redness, pulling of the scar, itching, restriction of joint movement, discomfort and cosmetic concerns)

Instructions for use:

- For optimum results preferably start using Scarban® silicone therapy immediately after wound closure (after removal of sutures and the disappearance of any scabs).
- It is recommended that Scarban® silicone therapy is build up gradually, up to the maximum of 23 hours of use per day. The purpose of building up gradually is to allow the skin to become accustomed to the silicone.
- For the best possible results, it is preferable that the silicone sheets are used for a minimum of 12 and a maximum of 23 hours per day.
- If necessary, Scarban® silicone sheets can be cut to the correct size (fig. 1).
- Ensure that the silicone sheet extends two centimetres beyond the edges of the scar.
- Ensure good adhesion and durability of the silicone sheet by removing any hair around the scar site.
- Thoroughly clean and dry the skin before applying the silicone sheet to the scar and the area surrounding it (fig. 2).
- Remove the protective film from the back of the silicone sheet (fig. 3) and apply Scarban® to the scar (fig. 4).
- If necessary, the silicone sheet can be fixed in place with an adhesive or elastic bandage (fig. 5).

- After removal of the silicone sheet, the skin may be treated with a hydrating cream.

Maintenance:

- It is essential to clean the Scarban® silicone sheets properly, on a daily basis. This is necessary to ensure good adhesion.
- Rub the silicone sheet with a few drops of Scarban® Silicone Cleaning Soap or a neutral soap (fig. 6).
- Remove as much dirt as possible with soap and lukewarm water.
- Rinse the silicone sheet thoroughly under lukewarm running water (fig. 7).
- After contact with soap and water, the silicone sheet will no longer be adhesive and it will roll up (fig. 8).
- Leave the silicone sheet to dry on a clean, dry surface, with the sticky side facing upwards (fig. 8).
- While drying, the silicone sheet will unroll and its adhesive strength will gradually return.
- Use the silicone sheet only after it has completely dried.
- Trim edges of silicone sheets that are no longer adhesive, due to wear and tear.

How long can I use Scarban®?

- It takes at least 3-6 months for a scar to normalise.
- Use the Scarban® silicone sheet as long as the scar is raised, red or causes complaints.
- Consult your physician about the duration of the therapy.

Duration of use:

- Each pack of Scarban® contains one or more silicone sheets.
- If used correctly and cared for properly (according to the instructions for use), each pack of Scarban® is sufficient for approximately 4 to 8 weeks of treatment.
- In the event of long-term therapy, several packs may therefore be necessary.

Special care and precautions:

- Keep out of reach of children.
- Scarban® silicone sheets are only suitable for external use.
- Use Scarban® silicone sheets only on healed, closed skin (after the scab has disappeared) and never on open wounds.
- Never dry Scarban® silicone sheets in the sun or near a heat source.
- Temporarily remove the Scarban® silicone sheet when exercising or swimming. Perspiration or contact with water may cause the Scarban® silicone sheet to detach.
- Do not use if you are hypersensitive to any of the ingredients.
- Do not use if the packaging is damaged when you receive the product.

Side effects:

- In rare cases, the skin may have a hypersensitive reaction (skin irritation) to silicone treatment. In such cases, stop the treatment until the skin is no longer irritated.
- Start the treatment again and let the skin slowly get used to the Scarban® silicone sheet (applying it for an additional hour every day).
- If skin problems persist, consult your physician.
- If there are serious adverse effects related to the product, these should be reported to the manufacturer and to the competent authorities (of the Member State) where the user is located.

Storage:

- The expiry date is stated on the packaging.
- Scarban® silicone sheets can be stored for 3 years in its unopened packaging.
- Store at temperatures between 5 and 25°C.

Date of last revision of instructions for use: June 2022

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